

Product Name :

Urine Test Strips

Product Code :

HALTH1090014

**Description :**

Urine Test Strips

Technical Specification :

The strip can be used to measure glucose levels in osmosis/diffusion experiments, urine analysis (educational only), and food science studies.

The Glucose test strip is specific to testing glucose (detects 0 – 3000 mg/dL or 0-3%) and is intended for educational use only.

Instructions:

Remove one glucose test strip from the amber bag.

Reseal the bag as soon as possible.

Do not touch the test pad with your fingers.

Completely immerse the test pad into specimen for 1-2 seconds.

Run the test strip along the edge of the container to remove excess liquid.

Compare to the color chart after 2 minutes for urine test, or after 3 minutes for water solutions of glucose.



Equipments Exporters

Website: www.equipmentsexporters.com, **Email:** sales@equipmentsexporters.com

Address: 75, Lajpat Nagar-IV, New Delhi-110024 **Phone:** +91-9311469084